### (EN) **blink®** refreshing hydrating eye mist 10ml

#### For relief from dry-feeling, tired or irritated eyes

**blink®** refreshing hydrating eye mist is a biocompatible lubricating, moisturizing and refreshing ophthalmic solution.

#### Indications:

Spray onto the open eye for relief from dry-feeling, tired or irritated eyes caused by prolonged computer use, driving, contact lens wear, insufficient tearing, smoky or air conditioned environments, exposure to wind and dust. Suitable for use with contact lenses.

#### Instructions for use:

- Hold the bottle firmly, without covering the spray opening, about 10cm directly to the side of your eye
- Look straight ahead, press firmly twice per eye
- Use as often as required
- If you wear contact lenses there is no need to remove them first

#### Symbols

Consult instructions for use	Manufacturer	LOT Batch code	Use by	25°C Temperature limitation
<b>STERILE A</b> Sterile. Aseptic technique	Do not use if package is damaged			

#### Warnings:

- In case of eye irritation, stop using the product and contact an eye care professional
- Do not use if you are allergic to any of the ingredients
- Do not touch the surface of the eye with the spray
- Keep away from heat sources, store in a dry place between 5 and 25°C
- Keep out of the reach of children
- Do not drink
- Do not use the product after the expiry date
- Use within 90 days of first use/spray
- Sterile until first use/spray
- Replace lid immediately after use

#### Contraindications:

None known.

**Composition:** Sterile isotonic, buffered solution pH 7.2 containing sodium hyaluronate, disodium edetate, polyhexamethylene biguanide and purified water.

Does not contain chlorhexidine, thimerosal, benzalkonium chloride, phenoxethanol.

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© Johnson & Johnson Surgical Vision, Inc. 2017
blink is a trademark of Johnson & Johnson Surgical Vision, Inc.